



Evacuation Stair Chair Bracket

Item#: 300TS/TSBLG

Wall mount bracket for the lightweight, manual and battery powered evacuation stair chair.

For Models:

- Aluminum Alloy EMS Evacuation Stair Chair (MS3C-300TS)
- Battery Powered Emergency Stair Chair (MS3C-300TSB)
- Lightweight Stair Evacuation Chair (MS3C-100TS)

Specifications:

- Includes rack and mounting hardware
- Heavy-duty high-strength polymer
- Measures 17-1/2in x 8in x 5in; supports up to 75 lbs
- Manufacturer warranty: 1 year

Installation Instructions:

1. Choose a safe, convenient location for evacuation chair storage. Position the chair rack at a height to ensure that the wheels of stair chair are firmly resting on the ground. Center mounting holes on studs with 16" (40.6 cm) centers. Drill 1/8" (3 mm) pilot holes prior to installing each of the screws in the following steps.
2. Tighten screw through one of the top corners of the chair rack into the center of the first stud.
3. Rotate the chair rack and make sure it is level, then mark and insert the second screw.
4. Fasten the chair rack to the wall with the remaining screws.

