



toll free: 1-877-706-4480  
international/local: 713-706-4480  
fax: 713-706-4161  
[www.MS3C.co](http://www.MS3C.co)

## MS3C- Stair Chair Second Operator / Lower Telescoping Handle Removal



1. Remove top bolt with Allen wrench on outside of bolt (red arrow) and adjustable wrench on nut on inside of chairs frame (blue arrow)



2. Pull out bolt. Nut between handrail and frame may need to be turned in order to release the bolt.




3. Once bolt is removed, telescoping handle will no longer be fixed to the frame. Push red release button (illustration 1). Remove plastic cap covering end of frame (illustration 2). Slide out handrail.



toll free: 1-877-706-4480  
international/local: 713-706-4480  
fax: 713-706-4161  
[www.MS3C.co](http://www.MS3C.co)



4. When handrail is extended you will notice a black insert under the rail (illustration 1). Pull hard while depressing the red release button and the guiderail will come out of frame channel. (illustration 2)

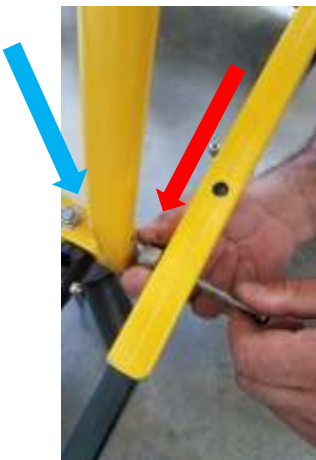

5. To secure telescoping guide bar channel to frame re-insert bolt. Make sure to place spacing nut between guide bar and frame (red arrow). Locking nut will be on the inside of the chair frame. (blue arrow)

6. To re-install the telescoping handle, reverse the steps. You will need to re-insert the guide bar, place the black insert under the guiderail making sure the bolt hole on the insert is lined up with the bolt hole on the guide bar channel. Depress red locking button, slide the handrail back in, make sure to slide black plastic cap. Re-insert bolt, spacing nut and locking nut.